® *Cyclotest*

Ovulationstest

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Cyclotest-Beratung
UEBE Medical GmbH

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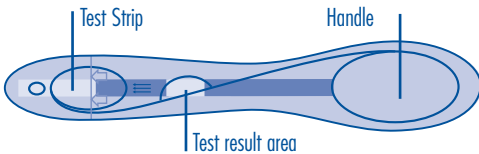
In-vitro-diagnostic device for determination of luteinizing hormone in the urine.

Notice: This users' manual includes additional information for Cyclotest 2 Plus users.


Cyclotest Ovulationstest

If you have decided to try for a baby, you want to know when your chances of conceiving are best. By measuring your body temperature regularly you will be able to define your fertile days. The most fertile days and therefore the best time for conception is indicated by the luteinizing hormone (LH), which can be detected with the help of Cyclotest Ovulationstest.

Please use the sticks only when the temperature measurement indicates that you are in your fertile phase.

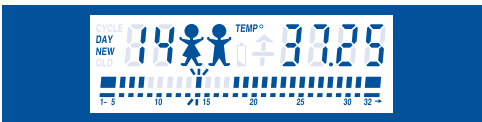If you are using Cyclotest 2 Plus you will recognize the start of your most fertile phase when the display shows baby symbols. On the basis of the stored data Cyclotest 2 Plus calculates this time and shows it in the display with two baby symbols, as i.e.:

In order to determine when you are most likely to conceive, you should now use the Cyclotest Ovulationstest.

This test provides you with reliable indications as to whether, and when, ovulation takes place. Ovulation is the moment at which the ovaries release a mature ovum (egg cell). For women of childbearing age, levels of the

hormone hLH (human luteinizing hormone) in urine lie between 3 and 15 IU/L (International Units per litre). About 24 hours before ovulation, this rises to levels as high as 200 IU/L. This test detects such a rise, indicating that conception can soon take place. Since sperm cells normally survive for several days inside a woman's body and take some time to migrate towards the ovum, this rise in hormone levels also marks the start of the phase in which sexual intercourse is most likely to lead to fertilization of the ovum. Then, between one and two days after ovulation, body temperature rises, marking the end of your fertile period.

Please note:

Read instructions carefully before use!

Use the test only once.

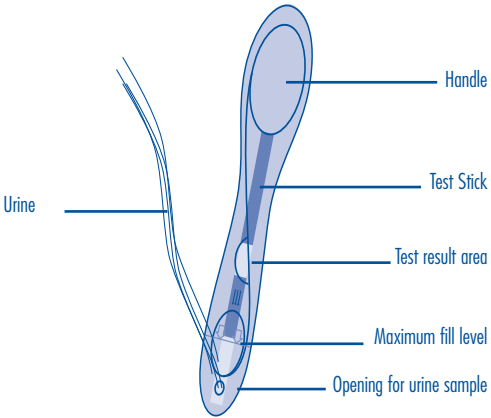
- Keep out of reach of children.
- Do not use the test after the expiration date and in case of damaged foil pouch.
- Test Components:
Box, instructions for use and foil pouch with test stick and desiccant.
- Store at 4°C to 30°C (39°F to 86°F)
- Please use the test immediately after opening the foil packing.
- Do not dismantle the test stick.
- For external use only.
- In-vitro diagnostic device for self-testing.

You can do this test at any time of the day, however the first urine in the morning is preferable because the concentration of the LH hormone is the highest in it. If you want to do the test during the day, you should not go to the toilet for 2-3 hours before the test and not drink very much.

This test conforms to Directive 98/79/EG of 1998 on in-vitro-diagnostics and bears the CE mark "CE 0123".

Easy application

Open the foil pouch just shortly before performing the test. Please prepare a watch with a second hand and eventually a clean container for your urine. Take the test stick out of the foil pouch. Discard the attached desiccant (a tiny white pouch), that protects the test from humidity, unopened with the regular household waste.

Hold the test stick at the handle with the openings for run off pointing downward to a light stream of urine for approx. 3 seconds (see image). The amount needed runs down the stick into the opening for the sample.

If you prefer, you can collect the urine sample in a clean and dry container. Dip the opening for the urine sample into the collected urine for about 3 seconds

Attention: observe the maximum fill level!

Put the test stick aside onto the foil pack for 5 minutes

Interpretation

If two purple lines appear in the result window and if the lower line is the same shade as the upper line or even more intense, then you are within your fertile time-period.

At this point you may want to discontinue testing and store the remaining test sticks for later use (if needed).

If two purple lines appear in the result window and the upper line is a significantly darker shade than the lower line, this means that the test was performed correctly but your fertile days have not yet started.

In this case you should perform additional tests on subsequent days until you obtain a positive test result.

The test is invalid if no purple line appears after 5 minutes. Reasons for obtaining an invalid result may be a damaged foil pouch, improper storage, or a mistake when performing the test. Please keep all components and contact UEBE Medical GmbH, Wertheim.

After 10 minutes the test results cannot be interpreted correctly anymore.

Dispose the test stick with the regular household waste.

IMPORTANT INFORMATION: This is an immunological test for the detection of human luteinizing hormone (hLH) in urine. Increased hLH levels in your urine mean that ovulation will probably (but not definitely) occur. Conversely, the absence of any rise of hLH does not always mean that ovulation will not occur. Test components of animal origin (such as antibodies) are potentially infectious material, but present no risk to health provided that all test components are used in accordance with instructions.

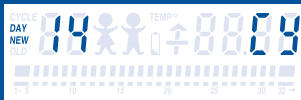
Additional Information

- This test is used to determine ovulation and thus your fertile days. However, since conception is influenced by many factors, it is not possible to assume that you will definitely become pregnant. If, despite using the test, you fail to become pregnant over a period of several months, please see your doctor.
- For most women with regular menstrual cycles, 5 testing sticks are sufficient. If your cycle is irregular, you may need a further test pack. You may occasionally have a monthly cycle without a rise in LH and hence no ovulation. If, after three consecutive cycles, and despite correct use of the test, you are not able to detect any rise in LH, please consult your doctor.
- Medicines containing hLH oder hCG may distort the test result. Similarly, false results will be yielded if you are already pregnant or in the menopause.
- This test is not suitable for contraception.

Entering the result into the Cyclotest 2 Plus:

If the luteinizing hormone (LH) has been detected the result data must be entered into the Cyclotest 2 Plus. Details you will find in the instruction manual.

Remaining Cyclotest test sticks still in their original packaging can be used – if necessary – in the next cycle.

Questions and Answers

How long can I read the test result?

The test results should be read after 5 minutes. A positive result will not disappear. It is useful to enter the result into your Cyclotest 2 Plus the same day, because your most fertile phase lasts only 1 – 2 days.

The stripe in the square test window of a negative result could become darker after exceeding the 5 minute limit.

How long can I store Cyclotest Ovulationstest?

If you store Cyclotest Ovulationstest in the closed foil bag between 4°C and 30°C, you can use them for an unlimited time up to the printed expiry date.

I have stopped taking the pill. How should I interpret my test results?

The human body takes a certain time to regenerate (normally 3 or more cycles). As soon as your cycle has become regular, you can use Cyclotest Ovulationstest.

Your Cyclotest advisory phone

Dear Customer,

The Cyclotest Ovulationstest is produced and tested with great care and under the most advanced procedures. If you need additional information, please do not hesitate to contact us. Our Cyclotest consultant team is at your full disposal for a confidential call. We reply to your questions personally and, of course, tactfully.

Cyclotest-Consultant
UEBE Medical GmbH
Zum Ottersberg 9
97877 Wertheim/Germany

 +49 (0) 93 42 - 92 40 40

info@uebe.com
www.cyclotest.de

Sincerely yours
Cyclotest Consultancy Team

P.S. If your wish to have a baby is fulfilled with the help of Cyclotest, we would be delighted to hear from you.

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© Copyright UEBE Medical GmbH