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DE

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Instructions for use

GB

TEARS AGAIN® Augenspray Sensitive – preservative-free liposomal eye spray with dexpanthenol

Ingredients:

Soy lecithin, sodium chloride, ethanol, dexpanthenol, vitamin A - palmitate, vitamin E, aqua purificata

Areas of application:

TEARS AGAIN® Augenspray Sensitive stabilises the protective tear film lipid layer and makes wearing contact lenses more comfortable. Liposomes and dexpanthenol moisturise the eye area and provide extra care as preventative protection for the skin.

For sensitive eye conditions such as dry or itchy eyes, burning sensations and the sensation of a foreign body.

Contraindications:

TEARS AGAIN® Augenspray Sensitive should not be used in the presence of hypersensitivity to any of the ingredients.

Directions for use:

Spray TEARS AGAIN® Augenspray Sensitive 1 – 2 times on the closed eyelids. Do not spray the product into the open eye or touch the eye or touch the eye surface with the spray head. Keep a distance of approx. 10 cm.

When using eye make-up, TEARS AGAIN® Augenspray Sensitive should be used before applying make-up and after make-up removal. In principle however, TEARS AGAIN® Augenspray Sensitive can also be used with previously applied eye make-up. In this case, only one spray should be applied and the distance between the dispenser and the eye should be slightly greater (about 20 cm).

TEARS AGAIN® Augenspray Sensitive makes wearing contact lenses much more comfortable.

General recommendation: use TEARS AGAIN® Augenspray Sensitive 3 to 4 times per day. For more severe cases, TEARS AGAIN® Augenspray Sensitive may be used more often.

Notes:

The bottle may also be used by several persons, since there is no contamination.

TEARS AGAIN® Augenspray Sensitive should not be used simultaneously with eye medicine. Should eye drugs other than TEARS AGAIN® Augenspray Sensitive be used, observe an interval of 10 minutes between the two applications.

It is important to check that the seal of the spray pump fits correctly and is undamaged before use.

The bottle is filled amply, because for technical reasons it cannot be emptied completely. It is guaranteed that the indicated quantity can be used.

Shelf-life and storage:

Store in a cool and dry place.

Keep out of reach of children.

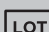
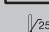
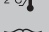
Can be used up to 6 months after first use.

Do not use after expiry date.

When using for the first time, you may have to pump 3 or 4 times to fill the tube.

Date of last revision: 2013-11

Symbol/Symbol/Simbolo/Symbol:

	Hersteller	Fabriqué par	Prodotto da	Manufacturer
	Verfalldatum (JJJJ-MM: Jahr-Monat)	Date de péremption (AAAA-MM: Année-Mois)	Data di scadenza (AAAA-MM: Anno-Mese)	Expiry date (YYYY-MM: Year-Month)
	LOT-Nummer	Numéro de lot	Lotto numero	LOT-number
	Lagertemperatur 2°C – 25°C	Température de stockage 2°C – 25°C	Temperatura di conservazione 2°C – 25°C	Storage temperature 2°C – 25°C
	Gebrauchsinformation beachten	Observer la notice explicative	Leggere attentamente il foglio illustrativo	Observe instruction leaflet
	Nicht verwenden wenn die Flasche beschädigt ist	Ne pas utiliser si le flacon est endommagé	Non usare se la bottiglia è danneggiato	Do not use if the bottle is damaged
	Nach erstem Gebrauch 6 Monate verwendbar	A utiliser dans les 6 mois après la première utilisation	Può essere usato fino a 6 mesi dopo il primo utilizzo	Can be used up to 6 months after first use