RC-CORNET[®] ADAPTER WITH MOUTHPIECE

To combine the respiratory therapy and inhalation therapy with the RC-Cornet[®] and the RC-Cornet[®]N

Please read the RC-Cornet[®] adapter with mouthpiece instructions for use carefully before first use. Please also follow the instructions of the nebulizer (the nebulizer is not included).

CEGLA
MEDICAL TECHNOLOGY

R. Cegla GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 2602 9213-0
info@cegla.de
www.cegla.de

INSTRUCTION FOR USE I

For the combination therapy of the lower respiratory tracts


The RC-Cornet[®] adapter combines the basic cornet with all common nebulizers.

Remove the mouthpiece of the nebulizer first. Now connect the RC-Cornet[®] adapter with mouthpiece to the nebulizer (see image).

Caution: The mouthpiece of the nebulizer must first be removed.

Then slide the adapter onto the RC-Cornet[®] (basic cornet) so that it fits tightly. Set the RC-Cornet[®] to the desired pressure profile by turning the mouthpiece at the bent end. Enclose the mouthpiece of the RC-Cornet[®] adapter with your lips, inhale and breathe in the aerosol and then exhale against the generated oscillating resistance.

Recommended frequency of the combination therapy: 2–3 x daily, 2–5 minutes (with severe mucus buildup more often).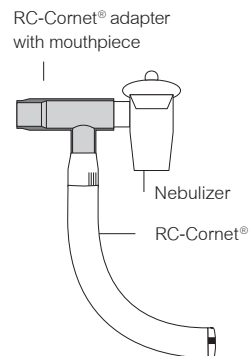

INSTRUCTION FOR USE II

For the combination therapy of the upper respiratory tracts

The RC-Cornet[®] adapter combines the nose cornet with all common nebulizers.

Pull the nose olive out of the RC-Cornet[®] N (nose cornet). Place it into the RC-Cornet[®] adapter and then add the converted adapter onto the cornet.


Connect the system to the nebulizer, so that it fits tightly (see image). Caution: The mouthpiece of the nebulizer must first be removed.

Set the desired pressure profile (setting positions 0–1) on the RC-Cornet[®] by turning the mouthpiece at the bent end.

Close off one nose hole with your pointer finger and push the nose olive of the RC-Cornet[®] adapter into the other nose hole.

Breathe in the aerosol through the nose olive and generate the healing pressure oscillation in the nose-throat area by exhaling.

Recommended frequency of use of the combination therapy: 2–3 x daily, 2–5 minutes (with severe mucus buildup more often).

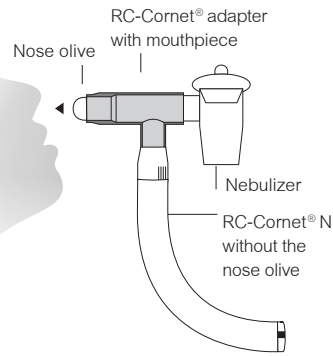CLEANING RECOMMENDATIONS

The RC-Cornet[®] adapter with mouthpiece can be cleaned as follows:

- In the RC-Clean cleaning bag for the micro-wave (sold separately)
- Boiling
- In a vaporizer
- With hot steam: temperatures up to 134°C

Clean the RC-Cornet[®] adapter with mouthpiece every 2 days!

The RC-Cornet[®] adapter with mouthpiece is meant for use by a single patient. For clinical reuse instructions for multiple patients, visit www.cegla-ltd.com

article-no. 225e | PPN 1101 4131 3578

Other products available in pharmacies, medical supply stores or at www.cegla-shop.com

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