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EINFÜHRUNG

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INSTRUCTIONS FOR USE

INTRODUCTION

GraviQUICK® is a pregnancy test that can be easily and independently performed at home. The test has a detection limit of 10mIU/mL.

GraviQUICK® is reactive towards the presence of the human hormone chorionic gonadotropine (hCG), which can be detected in urine at an early stage of an existing pregnancy.

During a pregnancy, hCG is formed in the female body and can be detected in urine. The pregnancy test contains antibodies, which specifically react with this hormone. Upon contact of urine with the test cassette or test stick (midstream), the sample migrates through the membrane by capillary action. Once the hormone hCG reaches the test area (T), a colored line forms. A negative result is indicated by a missing T-line. To ensure the successful performance of the test, a line will always be visible in the control area (C).

To determine and interpret the results correctly, it is necessary to carefully read the instructions. All detail should be understood before performing the test

MATERIAL

Cassette:

- 1 hCG-test cassette and 1 pipette in foil pouch with 1 pack of desiccant
- 1 instruction pamphlet

Midstream:

- 1 hCG-midstream test stick in foil pouch with desiccant
- 1 instruction pamphlet

Material required but not provided

- 1 timer
- 1 clean container (cassette test)

PREPARATION


GraviQUICK® works from the 10th day after conception.

Urine specimens collected at any time of the day may be used. First morning urine specimen is preferred due to its high concentration of hCG.

TEST PERFORMANCE

Test Cassette


1. Collect urine in a clean and dry container (not included).
2. Open the foil pouch by pulling the seal and remove the cassette from the pouch.
3. Using the pipette, draw some urine from the sample.
4. Holding the pipette vertically over the sample well (S), add **exactly 3 drops** to the device. **Please make sure not to add any liquid onto the result window – marked (T) and (C).** After applying the sample, do not touch or move the test cassette.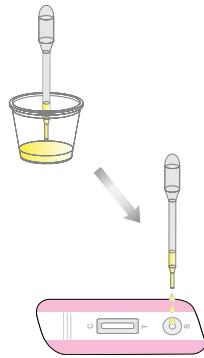

Read the result between 3-5 minutes after sample addition. Do not read the result later than 5 minutes.

Midstream test stick

1. Open the foil pouch by pulling the seal and remove the test device from the pouch.
2. Hold the handle of the test with one hand, removing the cover with the other. Place the cover in reach for closing the test again later.
3. Point the absorbent tip downward, and place it **in the urine stream for at least 10 seconds**. Alternatively, you can also collect urine in a clean container (not included), and submerge the tip of the test for at least 10 seconds.
4. Re-cap the device and wait for colored bands to appear.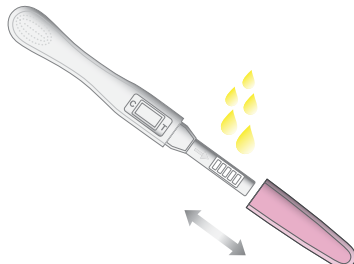


Read the result between 3-5 minutes after sample addition. Do not read the result later than 5 minutes.

INTERPRETATION OF RESULTS

Positive Result

If **two coloured lines** appear and the **test line (T) is the same or lighter than the control line (C)**, a pregnancy has been detected. The color intensity of the line can vary over the course of the pregnancy.

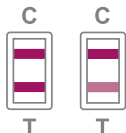
Negative Result

Only one colored line appears in the control area (C) but not in the test area (T). This indicates that no pregnancy has been detected.

Invalid Result

No line is visible in the control area (C): The result is invalid if **no colored line appears in the control area (C), even if a line is present in the test area (T).** Double-check if you followed every step of the instruction correctly. Repeat the test using a new device and urine sample.

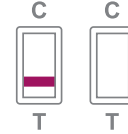CAUTION!

If the colored line in the test area (T) is very faint, we suggest you repeat the test within 48-72 hours using the first morning urine.

If the result is positive (two lines):

Please contact your physician to confirm the pregnancy using a clinical laboratory method. Under certain circumstances, false positive results are possible.

Your physician will provide additional information and help during your pregnancy.

If the result is negative (one line):

In case you still suspect a pregnancy (e.g. no menstruation), you may repeat the test after a few days. If a negative result is confirmed, please contact your physician.

PERFORMANCE DATA

GraviQUICK® detects hCG concentrations from 10mIU/mL to 500 000 mIU/mL in urine. The test result is not affected by the presence of lutenising hormone (LH, up to 300 mIU/mL), follicle stimulating hormone (FSH, up to 1000 mIU/mL) und thyroid-stimulating hormone (TSH, up to 1000 µIU/mL).

INTERFERENCE WITH THE TEST RESULTS

- False negative results can occur when the test is performed too early. For a reliable result, wait until one week after your missed period if possible. Higher levels of hCG can be reached in this time, rendering the test more accurate.
- Increased liquid uptake shortly before testing can lead to a false negative result. For most accurate results, use first morning urine for testing.
- Some fertility drugs which contain hCG can lead to false positive results.
- A range of medical conditions (e.g. ovarian cysts, chorionic carcinoma, tubal pregnancy) can lead to elevated hCG levels.
- Testing within 8 weeks of childbirth or miscarriage can also lead to a positive result.

WARNINGS AND IMPORTANT INFORMATION

- The test is intended for external use only.
- A final diagnosis must be confirmed by a gynecologist.
- Do not use the test for contraception/birth control.
- Do not ingest any part of the test.
- Keep out of the reach of children.
- Protect from sunlight, do not freeze. Store in a dry place between 4°C und 30°C.
- Do not use after the expiration date printed on the package.
- Not following the exact instructions can affect the outcome of the test.
- Do not use the test if the packaging is damaged.
- Do not reuse the test or test components.
- The test should be performed immediately or within one hour after opening the foil pouch.
- Poor vision, color blindness or poor lighting may affect your ability to interpret the test correctly.
- Do not touch the reaction window of the kit to prevent contamination.
- Each urine sample and used kit components should be considered potentially infectious. Contact with skin should be avoided.
- All test components can be disposed in household waste.

Explanation of symbols

Follow instructions	In vitro diagnostic medical device (for external use)	Best before (See imprint on package)
Store at 4-30°C. Do not freeze.	Content sufficient for 1 test	Do not reuse
Manufacturer	Catalogue number	Batchnumber (see imprint package)