Joyballs

JOYDIVISION
...my Lovestyle!

GEBRAUCHSANLEITUNG USER MANUAL

secret

Trend

Made with Love
in Germany

GUT ZU WISSEN

NICE TO KNOW

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Joyballs – for prevention, therapy or simply pleasure.

Joyballs are designed to actively assist in training the pelvic floor, to help increase sensation and strength. Joyballs can increase perception of the pelvic floor muscles and make the exercises more effective, hence achieving the desired results faster.

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Why should I train my pelvic floor muscles?

It is important for women to have strong and healthy pelvic floor muscles, as these keep the bladder, uterus, and bowels in their proper position. Only an intact and strong musculature can perform this task well. A pelvic floor weakness can have different causes: pregnancy, birth, menopause, overweight or the exercise of special sports.

Training strengthens the pelvic floor muscle and prevents weakening. During pregnancy, a strong pelvic floor can shorten and facilitate not only childbirth but also regeneration afterwards. Regular training with Joyballs can also improve the sexual sensation, which is reduced by weakened muscles. Targeted pelvic floor training with the help of Joyballs can alleviate existing discomfort or pain, prevent bladder weakness (incontinence) or organ lowering and achieve a better quality of life through strong pelvic floor muscles.

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How do I train with Joyballs?

Gently insert the Joyballs into the vagina (like a tampon). Especially in case of a dry vagina, the application of a lubricant (e.g. AQUAglide) can make insertion more comfortable. The Joyballs should be positioned deeply and almost vertically, with the cord facing downwards (the Joyballs secret feature the innovative integrated removal tab). A training session should last about 10 minutes and can be completed while standing or walking. At the beginning two training sessions per day should be sufficient. When you can hold the Joyballs for over 10 minutes easily, you should increase the duration of training and combine it with gymnastic pelvic floor exercises.

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How do the Joyballs work and how do I use them?

When your pelvic floor muscles are untrained, the Joyballs work on a „bio-feedback“ principle – a type of reflexive reaction of the muscles. The pelvic floor muscles grasp the Joyballs within the vagina, thus generating a training effect. This training effect is supported by the weight and anatomically adapted design of the Joyballs. The contractions executed du-

ring a training session follow the same principle as general weight trainings. The interior of the Joyballs contain balls that serve as weights; these oscillate against the outer wall with each movement. Thanks to the innovative, integrated „trampoline effect“, gentle „kinetic vibrations“ are set in motion, that additionally encourage your pelvic floor muscles to contract and relax as well as to provide you with stimulation.

Just as with other forms of training, pelvic floor training can leave your muscles feeling sore – so you should start slowly with the workout and then increase it continuously.

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What are the Joyballs made of?

All of our Joyballs are manufactured using Silikomed®, a material that has excellent skin-friendly properties and which has been clinically and dermatologically tested.

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All of the Joyballs can be used with water-based and silicone-based lubricants. Oil-based lubricants and cosmetics can degrade the Joyballs surface and are therefore unsuitable.

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Das Original.

Europas Nr. 1
Gleitgel-Marke
im Erotikmarkt.

Europe's No. 1
lubricant brand
in the erotic market.

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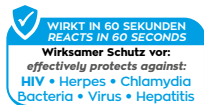
Before and after each use, clean the Joyballs with mild soap and water. Joyballs are also suitable for dishwashers.

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(EN) Guarantee

The guarantee on the material is 5 years for material defects, when the product is used for its intended purpose. The complaint process (sending in the item with sales slip) will be arranged by the retailer from which the article was purchased. The original sales slip must be retained in order for the guarantee to be given.

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When Joyballs should not be used?

When you are suffering from a vaginal infection, recovering from a wound (e.g. following an episiotomy) or during your menstruation. Following pregnancy, they should only be used after consulting your gynecologist or midwife.

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What do I have to consider?

Store your toy dust-free, dry and at room temperature. Please don't store with other silicone products or similar material. Please keep your JOYDIVISION products safely out of the reach of children. Do not use the products on diseased or irritated skin and do not pass on the products to third parties to prevent communicable diseases and infections. We assume no liability for damage and accidents caused by improper handling of our products.

SOLD AS NOVELTY ONLY

This product is intended for use as a novelty product and for adults only. Any product use for medical purpose or for a use that has an adverse effect on any function of the body is prohibited. Due to the nature of the product, we cannot accept returns or exchanges. For vaginal use only.

MADE IN GERMANY

JOYDIVISION international AG

Wiesenaue Strasse 11 · 30179 Hannover
GERMANY

www.JOYDIVISION.de

Distribution North America
JOYDIVISION (North America) LLC
12439 Magnolia Blvd. Suite 400
Valley Village, CA 91607 USA