 **VoltActive**

 gsk

 CE

## Sprunggelenkbandage / Ankle support

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	8
<b>FR</b> Instructions d'utilisation .....	13
<b>IT</b> Istruzioni per l'uso .....	18
<b>ES</b> Instrucciones de uso .....	22
<b>PT</b> Manual de utilização .....	27
<b>NL</b> Gebruiksaanwijzing .....	32
<b>SV</b> Bruksanvisning .....	36
<b>DA</b> Brugsanvisning .....	41
<b>NO</b> Bruksanvisning .....	45
<b>FI</b> Käyttöohje .....	50
<b>PL</b> Instrukcja użytkowania .....	54
<b>HU</b> Használati utasítás .....	59
<b>CS</b> Návod k použití .....	64
<b>HR</b> Upute za uporabu .....	68
<b>SK</b> Návod na používanie .....	72
<b>TR</b> Kullanma talimatı .....	77
<b>EL</b> Οδηγίες χρήσης .....	81
<b>RU</b> Руководство по применению .....	86
<b>ZH</b> 使用说明书 .....	91

	Seite Side	Größe Size	Umfang oberhalb des Knöchels Circumference above the ankle	
			cm	inch
	L/R 1)	XS	17-19	6.7-7.5
	L/R	S	19-21	7.5-8.3
	L/R	M	21-23	8.3-9.1
	L/R	L	23-25	9.1-9.8
	L/R	XL	25-27	9.8-10.6
	L/R	XXL	27-29	10.6-11.4
1) L-links/left; R-rechts/right				

<b>Material</b>	Gestrick/Knit fabric: PA, EL
	Frotteebezüge/Terry cloth covers: Baumwolle/Cotton, PES
	Pelotte/Pad: viskoelastisches Technogel®/viscoelastic Technogel®

## INFORMATION

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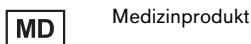
### INFORMATION

Date of last update: 2020-04-02

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Please contact the distributor if you have questions about the product or in case of problems.

- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

The instructions for use provide you with important information regarding the use of the VoltActive ankle support.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for orthotic fittings of the ankle and **exclusively** for contact with intact skin.

The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

The support must be used according to the indication.

### 2.2 Indications

- Irritation in the ankle (postoperative, posttraumatic and chronic)
- Swelling in the ankle (postoperative, posttraumatic and chronic)
- Slight feeling of instability
- Minor ligamentous instability
- Minor ankle pain
- Fibromyalgia
- Joint effusion
- Minor to moderate ankle sprain

Indications must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

Not known.

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the medical device will be applied; sensory and circulatory disorders in the foot area.

## 2.4 Effects

The form-knitted support with integrated shape-retaining pad provides ankle compression. The support promotes sensorimotor functions and thereby activates muscular stabilisation.

## 2.5 Conditions of use

The product is suitable for everyday and moderate sports activities, such as running, hiking, football, tennis, fitness training and gardening.

# 3 Safety

## 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

## 3.2 General safety instructions

 **CAUTION**

### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

 **CAUTION**

### Continuous use of the product

Changes in the state of health, worsening of the complaints

- ▶ Consult a physician promptly if you note any unusual changes on yourself.
- ▶ Do not continue using the product in case of doubt.

 **CAUTION**

### Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**NOTICE****Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

**NOTICE****Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

## 4 Handling

**INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.

### 4.1 Size Selection

The support is unisex.

- ▶ Select the support according to the side of the body, left or right.
  - 1) Measure the circumference above the ankle.
  - 2) Determine the size of the support (see size chart).

### 4.2 Application

**⚠ CAUTION****Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

**NOTICE****Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

The L-shaped pad marks the outside of the ankle.

- 1) Fold the support over so the heel can be grasped (see fig. 1).
- 2) Pull the support over the foot up to the heel (see fig. 2).
- 3) Fold back the section of the support that was folded over or unroll it upwards (see fig. 3).
- 4) Pull up the support so the ankle bones lie in the pad openings (see fig. 4).
- 5) Check the fit and correct positioning of the support.

### 4.3 Removal

- 1) Grasp the support with both hands on the sides and lower end of the textile material.
- 2) Pull the support down over the foot.

## 5 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the support regularly:

- 1) **Recommendation:** Use a laundry bag or net.
- 2) Wash the support in warm water at 30 °C with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 3) Allow to air dry. Do not expose to direct heat (e.g. sunshine, stove or radiator).

## 6 Disposal

Dispose of the product in accordance with national regulations.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

## 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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GlaxoSmithKline Consumer Healthcare GmbH & Co. KG