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IME-DC
Int. Medical Equipment Diabetes Care

IME-DC GmbH
International Medical Equipment – Diabetes Care
Fuhrmannstraße 11
95030 Hof | Germany
Tel: +49 9281 | 85 01 6-0 E-Mail: info@imedc.de
Fax: +49 9281 | 85 01 6-100 Web: www.imedc.de

PEN NEEDLES

Instructions for use

Compatible Pens

If your insulin pen is not listed, please contact IME-DC GmbH to check compatibility.

Berlin-Chemie	• BerliPen areo 3
Lilly	• HumaPen: Ergo II, Luxura, Luxura HD, Savvio • KwikPen: Abasaglar, Humalog, Humulin, Umluline
Novo Nordisk	• FlexPen • FlexTouch Pen • NovoPen: 3, 4, 5, Echo
Owen Mumford	• Autopen: 24, Classic
Sanofi	• AllSTAR • AllSTAR Pro • CliKSTAR • JuniorSTAR • SoloSTAR • SoloSTAR: Apidra, Lantus, Toujeo • TactiPen

Available sizes (PU 100 pcs.)

	REF	GTIN
4 mm	A068040DEEN-XX00X	08586015048050
6 mm	A068060DEEN-XX00X	08586015048067
8 mm	A068080DEEN-XX00X	08586015048074

Intended purpose

cleanFINE penta Insulin Pen Needles for single use are intended for delivery of insulin in conjunction with injection pens.

Intended users / patient population




Lay users (diabetics) with relevant knowledge of use these types of medical devices.

Contraindications

Medical device should not be used other than as indicated in intended purpose.

⚠ Warnings:

- Consult your healthcare professional for assistance in choosing the right needle length, injection technique and site.
- Before using pen needle consult and strictly follow insulin pen's instructions for use.
- Check the product and package integrity prior to use and also the expiry date.
- Do not use pen needles if the individual packaging is damaged or the cannula appears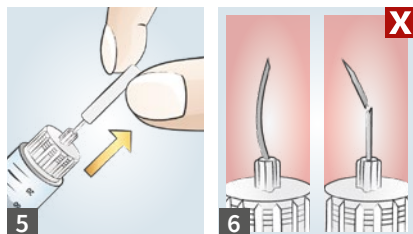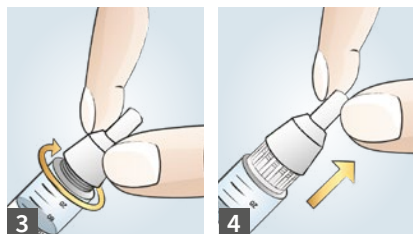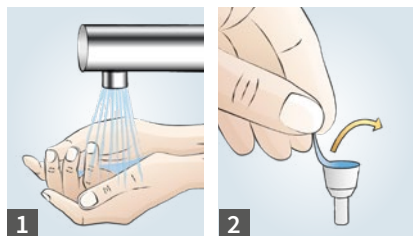




- bent or damaged (6).
- Remove both the outer cover and the inner needle cover before an injection. If both the outer cover and the inner needle cover are not removed before use, the medication or dose may not be injected, which may result in harm.
 - Do not re-use or share pen needles in order to avoid serious risk of infection.
 - Keep in dry place, away from sunlight and in temperature up to +40 °C.
 - Keep away from children of 3 years and younger. Contains small parts.
 - Report any health incident associated with the pen needle to a healthcare professional or the manufacturer.

General preparation (private user)

Wash hands with soap in lukewarm water, then rinse and completely dry (1).

Screwing the pen needle onto the insulin pen

1. Remove sterile paper from the pen needle (2).
2. Push the pen needle straight onto the insulin pen and turn clockwise until it is tight (3).
3. Remove the outer protective cap (transparent) and set it aside (4).
4. Remove the inner protective cap (white) by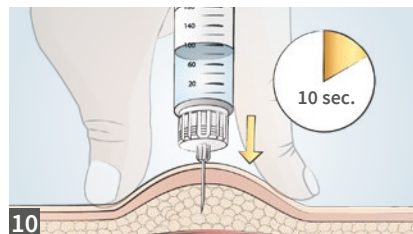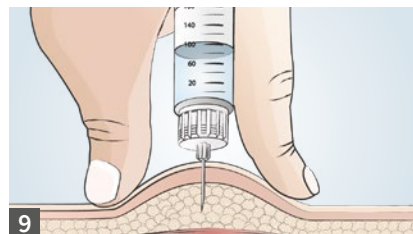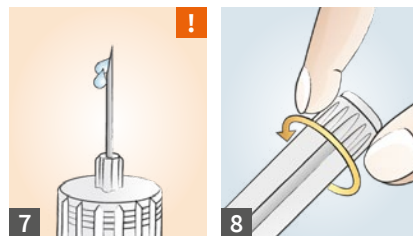



pulling it straight out to prevent bending of the cannula (5).

Preparation of pen needle and insulin pen

5. Check cannula to see if it is damaged (bent, broken, etc.) (6).
6. Prime the insulin pen as per the pen's instructions for use to ensure that pen needle allows free flow of medication (7). If no insulin flow is seen, place the outer protective cap (transparent) (12), remove the pen needle from the insulin pen (13) and discard the pen needle (14). Attach a new pen needle as directed above (2).
7. Set correct insulin dosage amount (8).

Carrying out an insulin injection (injection example)

8. Make skin fold and insert pen needle (9).
9. Inject insulin (slowly press down until button stops) according to instructions for use of the insulin pen (and instruction of your healthcare professional).
10. Leave pen needle in the skin for 10 seconds after injection (10). Do not change angle of the pen needle while it remains in the body, as this can result in breaking of the cannula (11).

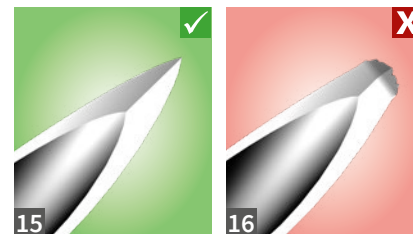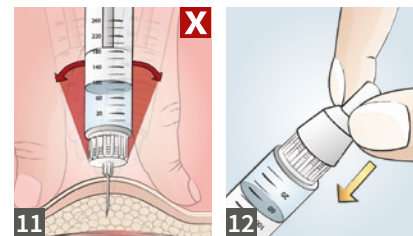11. Remove pen needle from skin while at the same time releasing the pinched skin.

After injection

12. Place the outer protective cap (transparent) onto pen needle (12).
13. Remove pen needle from insulin pen by turning counter-clockwise (13).
14. Discard used pen needle straight away according to your local regulations (14).

Pen needles are a one-time use product.

Do not reuse!

- New, unused pen needle (15).
- Reused pen needle: Quality and safety are reduced by multiple use of pen needles (16).

EXPLANATION OF SYMBOLS	
	Catalogue number
	Manufacturer
	Date of manufacture
	Consult instructions for use
	Use-by date
	Lot number
	Caution
	Keep dry
	Keep away from sunlight
	Do not reuse
	Do not use if the product sterile barrier system or its packaging is compromised.
	Latex-free
	PVC free
	Non-pyrogenic
	DEHP free
	Temperature limit -35 °C to +40 °C
	Sterilized using ethylene oxide
	Single sterile barrier system with protective packaging outside
	Medical device
	Unique Device Identifier
	Do not sterilize
	Distributor
	This product meets the requirements of the directive 2017/745

CHIRANA T. Injecta, a.s.
Nám. Dr. Schweitzera 194
91601 Stará Turá | Slovak Republic
Tel: +421/32/7709 991
www.t-injecta.com

PENCFP6-02-02/2022, Revision date 10.09.2022

Trademark Owner and Distribution

IME-DC GmbH
International Medical Equipment – Diabetes Care
Fuhmannstraße 11
95030 Hof | Germany
Tel: +49 9281 | 85 01 6-0 E-Mail: info@imedc.de
Fax: +49 9281 | 85 01 6-100 Web: www.imedc.de