# menoelle OVULA med

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### English

#### Vaginal Ovules

**menoelle Ovula med** is a medical device that helps maintain the natural defenses of the vagina and enhances the recovery the normal vaginal flora; containing hyaluronic acid, boric acid and lactic acid it helps delete the symptoms due to unbalanced vaginal hydration and/or vaginal pH. Hyaluronic Acid in its sodium salt form at a 10 mg/ovule dosage exerts an hydrating and lubricating action against vaginal dryness and contributes to restore a natural and comfortable vaginal environment. Lactic Acid, normally present in the vagina, favors an immediate pH correction and facilitates the natural recovery of the delicate vulvo-vaginal ecosystem. Boric Acid, thanks to its persistence in the vaginal environment, favors a lowering of pH prolonged over time.

#### Composition

One ovule contains: Boric Acid, Hyaluronic Acid Sodium Salt, Polycarbophil, Lactic Acid, Tocopheryl Acetate, Vitamin A, 18-beta-glycyrrhetic Acid, Tea Tree Oil, Semi-synthetic Triglycerides, Phosphatidylcholine.

#### Indications

It is useful in the prevention and treatment of vaginal dryness also in mycotic character. It helps reducing irritations, burning and itching.

#### Method of Use

- 1 Wash vaginal area properly (preferably with both a vaginal wash external and internal).
- 2 Open the package and after having washed properly your hands, take one ovule from the blister.
- 3 Gently insert the ovule deep into the vagina, preferably in supine position, avoiding accidental exit of the ovule out of the vagina.

### Dosage

Insert 1 ovule/day in the vagina – preferably in the evening before sleeping - for 5-7 days after the end of menstrual cycle or according medical prescriptions.

### Storage Conditions

Store the package properly closed in a dry place, between 5°C and 25°C, away from light and heat sources.

### Side Effects

Known hypersensitivity to any of the ingredients.

### Warnings

Keep out of reach and sight of children. Do not swallow. Do not use if package has been previously opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds. Do not use the product during menstruation. Do not exceed the suggested dosage without medical advice. The use - especially if prolonged - can lead to sensitiveness events. If this is the case, stop the treatment and consult the doctor in order to start a proper therapy. It is important to inform the doctor or the pharmacist of any undesired effects not reported in the insert leaflet.

### Precautions of Use

During pregnancy, lactation and for children the use of ovules must be limited only if strictly required and under direct control of your doctor. In case of the intact hymen, before using consult your gynecologist to the specific conditions of use. Disposable device. In case of accidental exit of the ovule out of the vagina, do not re-use the ovule.

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### Storage Conditions

Store the package properly closed in a dry place, between 5 °C and 25 °C, away from light and heat sources.

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### Warnings

Keep out of reach and sight of children. Do not swallow. Do not use if pack-age has been previously opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds.

Do not use the product during menstruation. Do not exceed the suggested dosage without medical advice. The use - especially if prolonged - can lead to sensitiveness events. If this is the case, pause the treatment and consult the doctor in order to start a proper therapy. It is important to inform the doctor or the pharmacist of any undesired effects not reported in the insert leaflet.

### Precautions of Use

During pregnancy, lactation and for children the use of ovules must be limited only if strictly required and under direct control of your doctor. In case of the intact hymen, before using consult your gyne-cologist to the specific conditions of use.

Disposable device. In case of the ovule accidentally exiting out of the vagina, do not re-use the ovule.

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