ocuers drops Relief

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Eye drops with sodium hyaluronate 0.21%

Composition

Sterile isotonic solution containing purified water, sodium chloride, potassium chloride, calcium chloride, magnesium chloride, sodium phosphate, citrate buffer, sorbitol and sodium hyaluronate 2.1 mg/ml.
Does not contain BPA (bisphenol A), phthalates nor borate buffer.

Preservative free**Form and content**

Sterile pump bottle with 10 ml

Indication

ocuers drops Relief is a preservative-free viscous comfort solution that lubricates and moistens the eye. ocuers drops Relief has been developed to induce natural tear film recovery, thereby preventing dry eye symptoms such as burning, runny, tired eyes and foreign body sensation. ocuers drops Relief can be used with contact lenses of any type of material.

Dosage, duration and method of application

- Contact your ophthalmologist or optician if you are requiring more than 10 drops a day.
- ocuers drops Relief can be used for up to 6 month after first use.
- Do not tilt your head back when applying ocuers drops Relief.

Important when using ocuers drops Relief 

- Do not use ocuers drops Relief if the bottle is broken or open in any way.
- The grip of the bottle may worsen if the solution comes in contact with the fingertips. Wash and dry the hands before further use.
- Do not use as storage solution for contact lenses.
- Avoid dropper tip contact with any surface.
- Store ocuers drops Relief in refrigerator or at room temperature.
- If the product is left unused for a long period of time or if you try to use ocuers drops Relief without the bottle being inverted to a vertical position with the tip pointing downwards, the pump needs to be primed again. To prime it, follow the above instructions for using ocuers drops Relief for the first time. Do not use after expiration date.
- **Keep out of reach of children.**
- When using ocuers drops Relief combined with other solutions, gels or ointments that are intended for ocular use, e.g eye drops, eye gels, eye ointments, contact lens solutions, or newly inserted contact lenses – wait a minimum of 30 minutes before applying ocuers drops Relief to your eyes.
- Do not use if you are allergic to any of the ingredients in ocuers drops Relief.

Side effects

Possible risks, side effects or complications connected to the use of eye drops are: allergic reaction, inflammatory reaction or infection. If you experience any problems contact your optician or ophthalmologist.

Medical device**Application**

1. Remove the protective cap.
2. Before using ocuers drops Relief for the first time, invert the bottle to a vertical position with the tip pointing downwards. Then, repeatedly press on the bottom of the bottle until the first drop leaves the tip. The bottle is now ready for use.
3. To dispense one drop of ocuers drops Relief first place the bottle between your thumb and index finger, with the tip of your thumb pressing against the ribbed surface around the tip, and the index finger on the bottom of the bottle. To reinforce your grip place the middle finger next to your index finger.
4. Without tilting your head backwards and with the bottom of the bottle pointing upwards, pull down the lower eyelid. Apply one drop into the inside of the lower eyelid, on the outer corner of the eye, i.e. away from the nose. Blink repeatedly to evenly spread the eye drop over the surface of the eye.
5. After use, make sure the tip is dry by flicking the bottle and then cover by putting the protective cap back on. To avoid contamination, never let the dropper tip come in contact with any surface.

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