Silver Nursing Cups

Instructions For Use

Indications

The device is a silver cup intended to cover and protect sore nipples during the breastfeeding period, between one feed and another. The protective barrier allows for drops of breastmilk to accumulate around sore skin which can support the body's own moisture wound healing.

Contraindications

There are no known contraindications.

Important safety information

Please keep away from extremes of temperature. Silver metal can conduct heat and reach very high or low temperatures.

- Do not use if you have a known or suspected allergy to silver or copper.
- Total usage of silver nursing cups should not exceed 3 consecutive weeks.
- If condition persists, discontinue use and seek the advice of a breastfeeding specialist or healthcare provider.
- Never leave packaging and/or components unattended.
- Keep out of reach of children when not in use. Danger of suffocation, if swallowed.
- For external use only.

Directions for use

- After your breastfeeding session, leave a couple of drops of milk in each cup (if desired) and apply them directly to the nipple.
- Remove cups and rinse nipples with water before breastfeeding.
- Do not use any other nipple treatment in conjunction with silver nursing cups, as this could reduce the effectiveness of the product.

Cleaning & care

- The device is not sterile and does not require sterilization/ disinfection before use and/or after each use.
- If you wish, rinse the silver cups with water after each use and allow them to air dry.
- Do not wash silver nursing cups in the dishwasher, as it may damage the silver.
- The silver cups can tarnish over time due to the natural oxidation process that occurs in silver, however it is still safe and effective to use.
- To remove the tarnish, make a paste using a mix of bicarbonate of soda (one teaspoon) with a few drops of water.
- Rub the paste over the cups, rinse, and pat dry.

Storing conditions: Product should be stored in a dry place away from direct sunlight.

Disposal method: This product is a medical device and should be discarded as a medical product according to your local procedures and regulations.

Should any serious incident occur in relation to the use of this device, please report this to the importer and your local competent authority.

SYMBOLS	MEANING OF THE SYMBOLS
	Consult instructions for use
	Single Patient – Multiple Use
	Do not use if packaging is damaged
	Keep the product dry
	Manufacturer
	Compliance with basic requirements of Regulation (Eu) 2017/745 of the European Parliament and of the council of 5 April 2017.

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Lansinoh Laboratories UK Ltd
Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | info@lansinoh.co.uk

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Lansinoh® is a registered trademark of Lansinoh Laboratories Inc. © 2025 Lansinoh Laboratories Inc. All rights reserved.

Health Plus (Shanghai) International Co., Ltd
Room 1-X3, Building 6, 4299 Jindu Rd
Minhang District, Shanghai, China 201108

EU	REP
----	-----

Authorized Representative
Riomavix Sociedad Limitada
(SRN: ES-AR-000001202)
Calle de Almansa 55, 1D, Madrid 28039 Spain

UK Responsible Person
LLINS Technology Ltd, Cambridge Innovation Park
Denny End Road, Waterbeach, Cambridge, Cambridgeshire
CB25 9PB England, United Kingdom

Date of issue of the instructions for use: 05/05/2025
SC65300IF0525 v2